

OBSERVED HEAT RECOMMENDATIONS FOR **SexedULTRA4M** SEMEN USAGE



14-20 hrs

A.I. with sexed semen **14-20 hours** after first signs of standing heat



Use temperature regulated thawing units

Regulate



Change thawing unit water daily

Daily



Clean thawing unit weekly

Weekly



35° to 37°C

Monitor water temperature with thermometer: **35° to 37°C**



Thaw for approximately **45 seconds**



45 sec



35° to 37°C



Keep semen temperature regulated at **35° to 37°C** with A.I. gun warmer



15 min

Use thawed semen within **15 minutes** of initial thaw

SexedULTRA4M™